




Safety facial shield #3

Mounting instruction




1

Safety face shield #3 and components

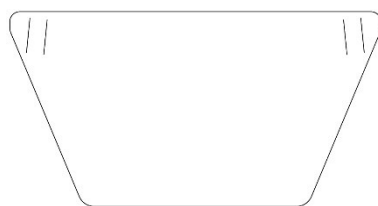

1 x Face Shield

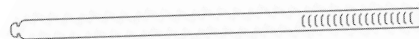

1 x Fastening Strap

2

Put the fastening strap through the set of holes in visor's corner




3

Pull the strap along visor's edge




4 Put the fastening strap through the set of holes in the opposite corner

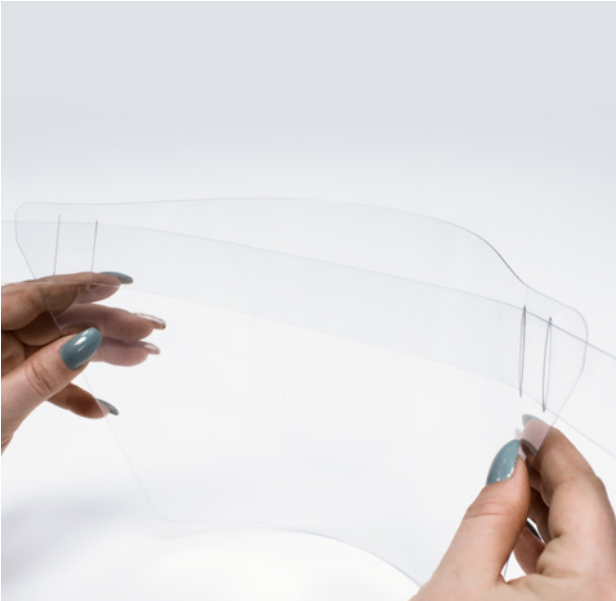

5 Adjust the strap to head's size by putting the hook through the chosen cut



6 Adjust the visor's face distance

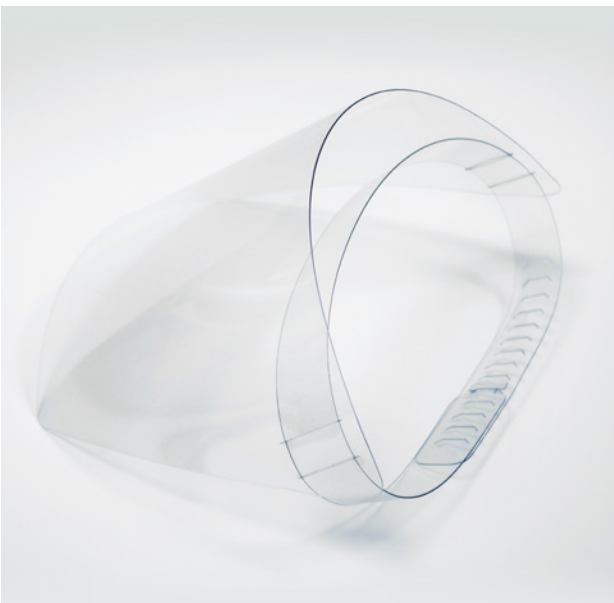

Unsterilised Product



Single-Use Product



Use at temperature of 5 – 30 °C

HL DISPLAY AB
Cylindervägen 18, Box 1118
131 26 Nacka Strand - Sweden

Tel +46 8 683 73 00
info@hl-display.com

www.hl-display.com

